

Genurine™ Reagent

Unsurpassed urine confirmation test



Cat. No. ACC001 (1L)


PRODUCT INFORMATION SHEET

Features:

- Test for the detection of real urine from synthetic or dehydrated urine.
- Simple, fast, one-step procedure for ensuring a sample is real urine before assessing the presence of drugs or banned substances.

Description:

The Genurine™ reagent offers a one-step test for the detection of real urine from synthetic or dehydrated urine. The Genurine™ reagent is specifically formulated to produce a precipitate that coats the glass vials when exposed to real urine. No precipitate coating on the wall is observed with synthetic or dehydrated urine.

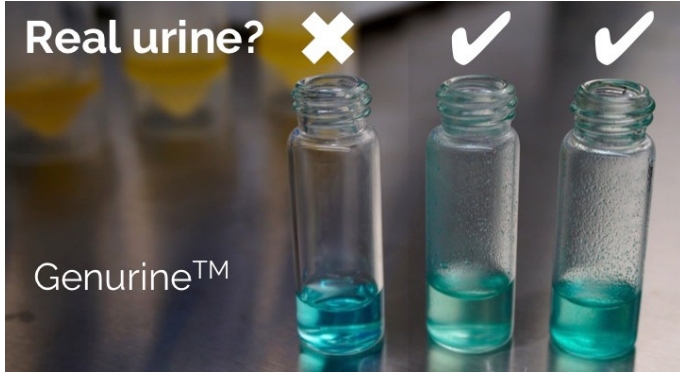

Kit Contents:

	Qty.	Storage Temp.
Genurine™ Reagent	1L	Room temp.
Product Info. Sheet	1	-

Protocol:

1. Dispense 600 µL Genurine™ reagent from 1L stock bottle to 4 mL glass vial.
2. Add 150 µL unknown test sample to the Genurine™ reagent in the glass vial prepared above.
3. Place the cap on the glass vial, tighten the cap and briefly shake the mixture.
4. Let the mixture sit for 20~60 seconds and observe the vial for the presence or absence of precipitate on the walls of the glass vial (see Representative Results).

Performance Characteristics:

Sample Type	Unique Samples or Lots	Replicates / Unique Sample or Lot	# of Times Called Real Urine
Fresh human urine	157	2	312
Test Clear (dehydrated urine)	3	13	0
Golden Shower (dehydrated urine)	3	14	0
Ultra Pure (synthetic urine)	3	20	0
Magnum (synthetic urine)	3	16	0
Quick Fix (synthetic urine)	3	18	0

Table 1. Genurine™ Reagent is able to correctly identify real and synthetic or dehydrated urine products with approximately 100% accuracy. Human urine that is diluted more than 2-fold with water may not be called “real urine”.

Note: ™ Trademarks of Entopsis, LLC. For Research Use Only (RUO). Not for use in diagnostic procedures. Manufactured in a FDA registered and ISO13485 / ISO9001 certified facility (7600 NW 69th Ave, Medley, FL 33134). Follow safety guidelines and rules enacted by your institution and country. A commercial license must be purchased from Entopsis LLC if this product is going to be re-sold within or outside of the United States of America.

Genurine™ Reagent


Unsurpassed urine confirmation test




Cat. No. ACC001 (1L)

Representative Results


Consistent with Real Urine:

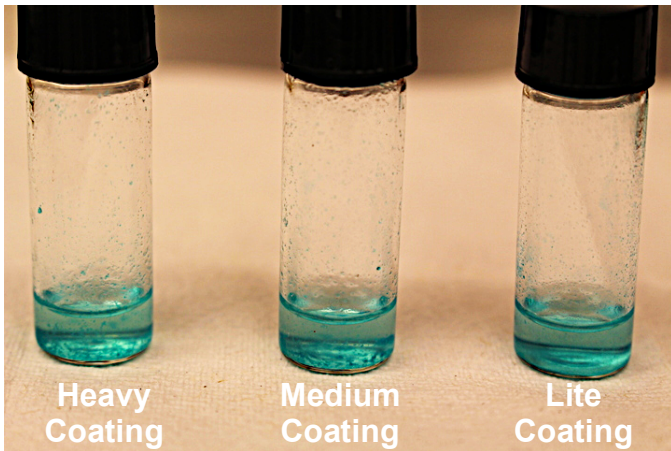

Not Consistent with Real Urine:

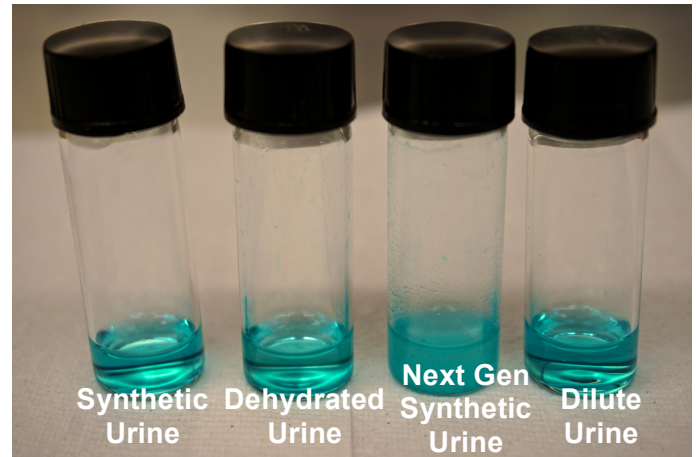

Examples of Real Urine Containing Blood:

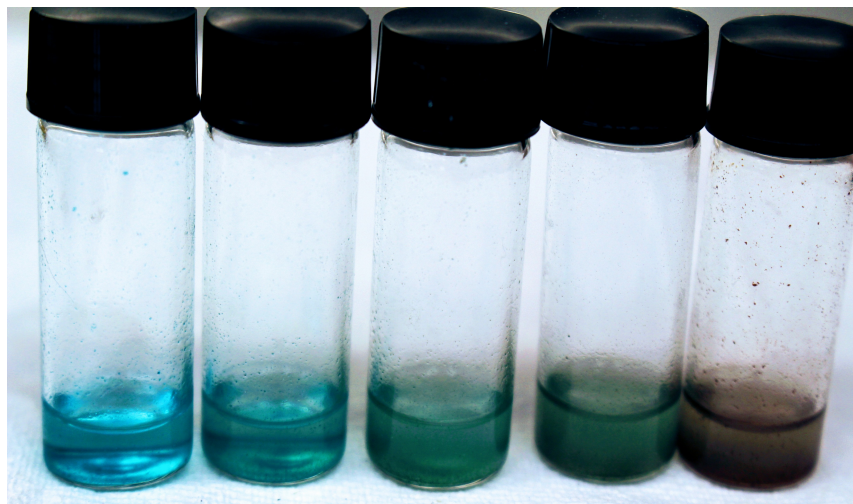

Note: ™ Trademarks of Entopsis, LLC. For Research Use Only (RUO). Not for use in diagnostic procedures. Manufactured in a FDA registered and ISO13485 / ISO9001 certified facility (7600 NW 69th Ave, Medley, FL 33134). Follow safety guidelines and rules enacted by your institution and country. A commercial license must be purchased from Entopsis LLC if this product is going to be re-sold within or outside of the United States of America.